

XSEAL LC

Light Curable Temporary
Filling Material



XSeal LC is a light curing flowable temporary filling material. It is made for a temporary filling of the implant abutments, but can be used as a sealing material for any small pit and cavity.



APPLICATION RANGE:

1. Implant abutments temporary sealing.
2. Temporary filling of cavities.
3. Useful for inlay / onlay cases.



PASSIONATE ABOUT REGENERATION

🌐 clinic@surgicalesthetics.com

📞 833-701-5245

✉️ www.surgicalesthetics.com

APPLICATION METHOD

BEFORE THE APPLICATION:

- Check the expiration date and avoid package contamination.
- Read the instructions before first use of the product.
- Open the syringe cap and place a disposable tip on the syringe.

APPLICATION:

- Prepare a cavity.
- Extrude the filling material by pressing the plunger

of the syringe.

- Apply the filling material to the fully fill the cavity.
- Light cure for 30 seconds.
- 5. The sealing plug may be removed with an explorer when necessary.

AFTER THE APPLICATION:

- Close the safety cap immediately after use.
- Store Once-Fil in a cool and dry place.
- Avoid direct sun-rays and strong ambient light.

PRECAUTIONS AND WARNINGS

- The material must be handled by dentists and dental specialists according to the instructions before using it.
- Use of DSI Dental Dam is recommended for all the patients to minimize the tissue contact.
- Do not use to patients who have rash on their skin, dermatitis or history of hypersensitivity to acrylates, silicones, or other resins.
- If allergic reaction, dermatitis or rash develops, consult a physician.
- Might be sensitizing. Avoid repeated contact of uncured dental resin with skin.
- If accidental eyes contact occurs, wash immediately with a plenty of water and seek medical attention.
- To avoid cross-contamination, do not re-use tips.
- Protective device for the respiratory tract is highly recommended before a surgical operation.
- Keep refrigerated for long-term storage to maintain shelf life.

STORAGE

Store in dark and cool place 1~30 °C. Avoid exposure to direct light. Keep out of reach of children.

SKU	VOLUME
XSLC05	2 grams
5 PER PACK	



PASSIONATE ABOUT REGENERATION